

# ColoSignal™

## One Step iFOB Test

For immunological detection of fecal occult blood

### Patient Instructions

If you have questions after reading all instructions, please contact your physician's office or laboratory for assistance. Or e-mail to Metas Biomedical Inc. at [service@metasbiomedical.com](mailto:service@metasbiomedical.com)

**IMPORTANT**

DO NOT collect stool specimen during or within 3 days of menstrual period.


Dietary restrictions are not necessary. Normal Diet (e.g. red meal and vitamin c) is fine.

DO NOT use test kit beyond the expiration date printed on the specimen collection tube.


DO NOT collect stool specimen if you have bleeding hemorrhoids or blood in the urine. Please contact your physician instead.

Follow additional instructions given by your physician, if applicable.


Before starting collection procedures, please make sure this kit contains all necessary items.

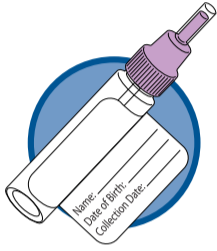


Specimen Collection Tube with attached Patient Label

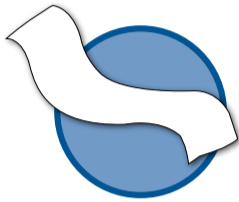

Collection Paper

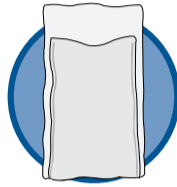

Absorbent Sleeve

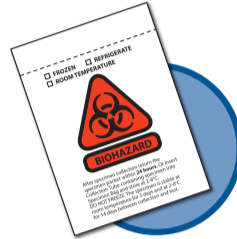

Specimen Bag



Return Mail Box

V2. 201607

## 1 Specimen Deposit



**A** Urinate first if possible. Flush the toilet bowl several times, and remove all toilet cleaners, disinfectants or deodorizers.



**B** Remove tape cover from each end of the Collection Paper, and adhere the paper to rear half of toilet bowl as shown. Allow Collection Paper to sag just above water.




**C** Lower the toilet seat, make bowel movement on Collection Paper. If need to urinate, then shift body forward on the toilet seat so as to avoid urinating on the Collection Paper.


## 2 Specimen Collection




**A** Fill out the attached label, then remove the back paper, and attach the label to the Specimen Collection Tube.

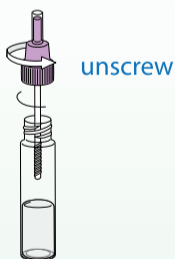

**B** Unscrew the cap from Collection Tube, and keep it in a vertical position. DO NOT pour out the buffer that is inside the tube.

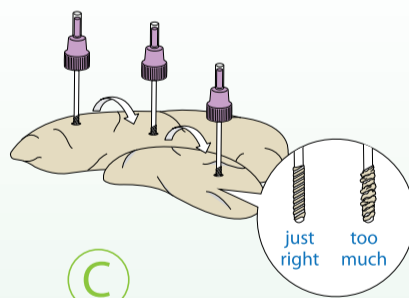


**C** Poke spiral applicator into stool at least 3 different sites to collect a moderate amount of stool specimen.

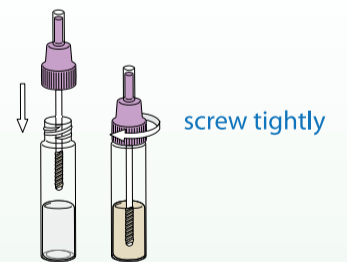


**D** Insert and screw the applicator back into the Collection Tube and secure tightly. Flush away the Collection Paper and remaining stool.


## 3 Specimen Return

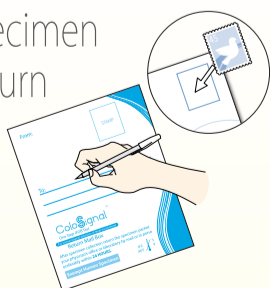


**A** Complete the "To" and "From" addresses, if blank. Apply stamps required for mailing.

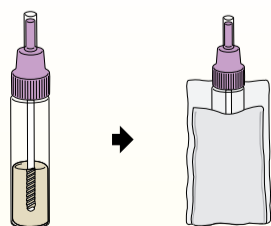

**B** Insert Collection Tube into Absorbent Sleeve, then insert Absorbent Sleeve into Specimen Bag and seal.

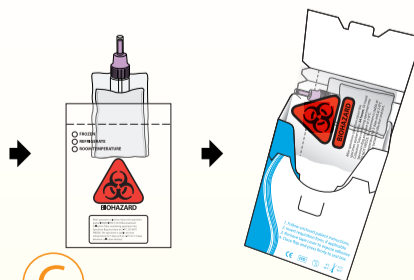

**C** Insert Specimen Bag into Return Mail Box and seal. If applicable, also put requisition form into Return Mail Box.

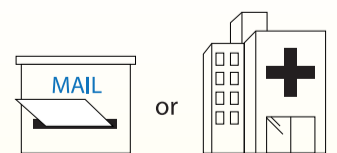

**D** Return the specimen packet to your physician's office or laboratory by mail or in person, preferably within **24 Hours** after specimen collection. **DO NOT mail to Metas Biomedical Inc.**